

What is it?

The Tunstall MedMinder™ medication reminder and dispensing system is designed to enhance medication adherence. The pill dispenser offers users a sense of comfort knowing that their medications are conveniently organized and that they will be gently prompted to take them at the appropriate times.

How does it work?

Dispensing times, reminder and notification preferences are easily set-up by a family member or caregiver via the convenient MedMinder portal. A family member, caregiver or pharmacist would then fill the dispensing cups.

When it's time to take a medication, the appropriate compartment flashes to prompt the user. The user can also receive audible reminders. If the cup is not removed within the assigned time frame, designated recipients will receive automatic notifications via phone, text, and/or email.

The MedMinder system is cellular enabled eliminating the need for a traditional phone line while allowing for remote programming and updates from the MedMinder portal.



The MedMinder system is designed with ease of use in mind, it's simple and friendly interface has no digital readouts or buttons. The MedMinder system contains 28 large pill compartments which can accommodate an array of pill types and can be configured for multi-week dispensing.

MedMinder provides rich reports regarding the medication activity. Detailed reports show when cups were taken out of the medication dispenser, when refilled and if any dispensings were missed.

Who is it for?

A wide variety of subscribers can benefit from the MedMinder, including people who:

- Have multiple prescriptions
- Have difficulty organizing pills
- Have a hard time remembering
- Have dexterity issues
- Have difficulty reading labels
- Have a risk of mis-dosing

The MedMinder portal offers users and caregivers the convenience of updating regimens and configuring the type and delivery method of notifications. Detailed reports are also available on the portal.

Key Features Benefits

- Cellular connectivity
- Customized reminder prompts
- Tamper proof lock
- 28 large dispensing compartment cups
- EZ refill trays
- Dispensing compartments only open at the designated time
- Remote programming
- Phone, email, and text notifications
- Cloud based management portal
- 12 hour battery back-up

- MedMinder uses as built in cellular modem to report activity and events. A traditional phone line is not required.
- The MedMinder system can be programmed to blink, beep, or play a customized message to remind the user at designated dispensing times. Messages can be selected from default options or be pre-recorded by a caregiver or provide familiarity or avoid confusion.
- The MedMinder device is lockable, which safeguards the medications from abuse or theft.
- Easily accommodates 10-12 standard pills. The 28 compartments allow flexibility for dispensing up to 4 dispensing per day for 1 week, 2 times per day for 2 weeks, or 1 time per day for 28 days.
- Refill trays make dispensing more convenient, and reduces the possibility of medication errors. Refill trays can be filled ahead of time by family members, a pharmacist, or caregiver, and placed directly into the MedMinder medication dispenser.
- All dispensing compartments are locked and only the appropriate compartment will open for each dispensing time which helps to prevent mis-dosing or overdosing.
- Users or authorized caregivers can remotely configure or update dispensing times, reminder prompts, and notifications via the convenient MedMinder Portal.
- Caregivers can be notified of pertinent events (ex. missed dose, refill request, loss of power) in their preferred communication medium(s).
- The MedMinder Portal allows users or caregivers to remotely program the system and access detailed device activity reports including compliance information.
- Ensures access to medications and delivery of reminders and notifications in the event of a temporary loss of power.

Specifications

• Part(s) for this solution	• MedMinder (100-200)
• Dimensions	• 13.75" x 10.75" x 2.25" (WxLxD)
• Weight	• 4.2lbs
• Battery	• 12 hour back-up battery (rechargeable)
• Installation Requirements	• AC power outlet AT&T 3G cellular coverage*
• Options	• Caregiver portal
• Maintenance Requirements	• Periodic cleaning

For additional information please call
1.800.286.2622
 or email US.Info@tunstall.com
 36-36 33rd Street, Suite 103, Long Island City, NY 11106
americas.tunstall.com

*AT&T is a registered trademark of AT&T™
 MA-16-0209S Rev 2.0

